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INSTRUCTIONS FOR USE

COLLAR STARK HALO – Halo brace

The COLLAR STARK HALO stabilizer for adults is intended to be used with COLLAR STARK only. Using the stabilizer properly is important for the patient immobilization and comfort. The two-part structure consists of the front part and the back part that are included in the set.

PUTTING ON:

1. Adapt the Halo Stabilizer to the plastic reinforcement in the back part of the collar by inserting it from the top and attaching the stabilizer to the occiput supporting element. Align both parts.
2. Having adjusted the Collar properly, wrap the strap around the patient's head over the centre of the forehead.
3. Adjust both straps at the same time using the same pressure and tension force. Pass the straps through the buckles and fasten them.
4. Make sure that all the stabilizing elements fit closely to the patient and that the patient has adopted a natural and correct posture.

Additional hook-and-loop fasteners included in the set are to be attached to the collar and the HALO band for better fixing.

TAKING OFF:

1. Before taking the HALO and Collar off, mark the hook-and-loop strap position to ensure proper fit when you put them again.
2. Hold the HALO and Collar, loosen the HALO strap at the right and then at the left side.
3. In order to take the Collar off, unfasten the hook-and-loop fastener at the right and then at the left side.
4. Remove the HALO and Collar from the patient.

SIZE: UNIVERSAL

THE FINAL DECISION REGARDING THE APPLICATION AND SELECTION OF EQUIPMENT IS MADE BY A SPECIALIST PHYSICIAN.