Gwarancja

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Materiały tekstylne: Poliamid, Lycra, Elastan.

Guarantee

The guarantee lies within the framework of the legal regulations.

Disposal

The product can be disposed of with the normal household waste.

Storage information

The storage location should be dry and at room temperature. The product has to be protected against direct sunlight exposure and humidity.

Cleaning information

The lateral coil springs remain in the bandage.
Please hang up the bandage dripping wet.

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Genu Meyra bandage with silicone ring

Thank you very much for selecting a product of the Meyra-Ortopedia Group. The product complies with all legal regulations according to the Medical Products Law.

Please read the directions for use carefully and keep to the advice and recommendations given. Should you have further questions please address the responsible physician, your local specialist dealer or our service hotline.

Product description

The Genu meyra active knee bandage is a highly effective medical bandage on the latest technical level. The anatomically bi-elastic knitted form offers a very good fit, lateral steel springs provide safety and stability. The bound round elastic silicone ring increases the compression, supports the patella and provides steady support. When moving it has a compression and massage effect on the surrounding tissue. This improves the blood circulation which contributes to the detumescence of the tissue, alleviation of pain, function improvement and relief at the same time.

Safety instructions

The first application of the Genu meyra active knee bandage must be carried out by special personnel trained on these kind of products as otherwise the product liability can not be guaranteed.

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Employment according to purpose and indications

The Genu meyra active knee bandage is used in case of slight instabilities and malpositions, arthritis, in case of attrition at the knee joint and chronic irritations.

Application of the bandage

- 1. Take the bandage at the lateral steel springs from inside and outside and pull it over the knee until the gel pad surrounds the kneecap.
- 2. Check the correct fit and adjustit if necessary.

Contraindications

Provided that the product is employed properly contraindications are not know. If the bandage is too tight pressure symptoms may evolve. In case of the following clinical pictures the product must only applied after agreement with the responsible physician: skin changes or inflammations in the area concerned, scarring, hypersensitivity because of heat production, varicose veins, circulatory disturbance in the legs, lymphatic drainage disturbance.

Application duration

The bandage should mainly be worn during physical activity as the function only works in this state. When resting the product should be removed.

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Further information

- The active bandage should only be worn under the care of a physician and for the intended clinical pictures.
- The Genu meyra active bandage is intended for the treatment of one patient.
- Improper application excludes the product liability of the manufacturer.
- Should the symptoms aggravate a physician should be consulted immediately.
- A combination with other products has to be agreed with the physician in advance.

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