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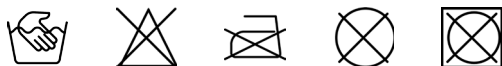
## Storage information

The storage location should be dry and at room temperature. The product has to be protected against direct sunlight exposure and humidity.

## Cleaning information

Please hang up the bandage dripping wet.

## Material

Textile: Polyester, lycra, elastane.

## Guarantee

The guarantee lies within the framework of the legal regulations.

## Disposal

The product can be disposed of with the normal household waste.

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### Malleo Medical ankle joint bandage with gel pads

Thank you very much for selecting a product of the Meyra-Ortopedia Group. The product complies with all legal regulations according to the Medical Products Law.

Please read the directions for use carefully and keep to the advice and recommendations given. Should you have further questions please address the responsible physician, your local specialist dealer or our service hotline.

### Product description

The Malleo medical ankle joint bandage is a highly effective medical product on the latest technical level. The anatomically bi-elastic knitted form offers a very good fit the integrated lateral elastic gel pads exert compression on the surrounding tissue which decreases the symptoms of the muscular and tendon insertion.

### Safety instructions

The first application of Malleo medical ankle joint bandage must be carried out by special personnel trained on these kind of products as otherwise the product liability can not be guaranteed.

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Employment according to purpose and indications

The Malleo medical ankle joint bandage is used in case of slight instabilities of the ankle joint, distortions, arthritis and arthrosis as well as posttraumatic and postoperative irritations. Any other employment, i.e. departing from the direction, results in an expiration of the product liability.

Application of the bandage

1. Turn the upper edge of the bandage slightly down to pull it easier over foot and heel.
2. Then turn the bandage up and check if the lateral gel pads enclose the ankle.
3. If necessary, readjust the fit of the bandage.

Contraindications

Provided that the product is employed properly contraindications are not known. If the bandage is too tight pressure symptoms may evolve. In case of the following clinical pictures the product must only be applied after agreement with the responsible physician: skin changes or inflammations in the area concerned, scarring, hypersensitivity because of heat production, circulatory disturbance in the foot/lower leg, lymphatic drainage disturbance.

Application duration

The bandage should mainly be worn during physical activity as the function only works in this state. When resting (sleeping) the product should be removed.

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Further information

- The Malleo medical ankle joint bandage should only be worn under the care of a physician and for the intended clinical pictures.
- The product is intended for the treatment of one patient.
- Improper application excludes the product liability of the manufacturer.
- Should the symptoms aggravate a physician should be consulted immediately.
- A combination with other products has to be agreed with the physician in advance.

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