DE

## ANWENDUNG:

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ENG

**ASSINGMENT:**

Trekker Universal is a long wrap-around brace of multifunctional, therapeutic use. It effectively supports treatment and rehabilitation for injuries and dysfunctions around the knee joint, in cases not requiring immobilisation.

**INDICATIONS:**

- Repetitive, subacute pain related to strain injury and micro-injury of muscles and ligaments.
- Gonarthrosis and knee joint instability.
- Stabilization after injury during the final phase of functional treatment.
- Injury and dysfunction which do not require knee.
- Immobilization (lateral ligament injuries).

**CONTRAINDICATIONS:**

- Conditions which require cast immobilization.
- Long-healing wounds and ulcers.
- Significant bone deformations (brace cannot be adjusted to the joint).
- Lower extremity varicose vein.

**PRODUCT ADJUSTMENT:**

- Open the cover on each ROM hinge and set the desired level of flexion/extension by placing the hinge pins in the appropriate degree position. Make sure that the flexion/extension settings are the same for each hinge, as failure to do so could result in poor performance from the brace.
- Place the brace around the knee so that the hinges line-up with the center of the knee cap on both sides of the leg, and secure the top closure.
- Secure the bottom closure using the Velcro tab provided. Make sure the hinges still align with the knee, and the brace fits snug so that little movement is experienced.
- Feed the top closure strap through the corresponding D-Ring and tighten to secure a snug fit.
- Feed the bottom closure strap through the corresponding D-ring and secure. Tighten and re-adjust as needed.

**ATTENTION!**

The application instructions listed above are intended for the use of qualified professionals only. It is not recommended for patients to attempt to apply or adjust this device unless otherwise instructed to do by an orthopedic professional. Any questions regarding further adjustment or frequency/ duration of wear should be directed to your orthopedic professional. as they know the individual condition for which this device was prescribed.

SIZE: UNIVERSAL

**CLEANING INSTRUCTIONS:**

Remove hinges from brace and hand wash in cold water with mild detergent. Allow all parts to air dry before reassembly. Do not dry clean, iron, or bleach any part of this brace, as it could cause damage and void the manufacturer's warranty.

THE FINAL DECISION CONCERNING THE APPLICATION AND CHOICE OF THE EQUIPMENT BELONGS TO A MEDICAL SPECIALIST.