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### Storage information

The storage location should be dry and at room temperature. The product has to be protected against direct sunlight exposure and humidity.

### Cleaning information

Clean the brace with moist cloth using mild detergents.

### Material

Textile: Polyester, elastane, cotton, silicone, elastomer.

### Guarantee

The guarantee lies within the framework of the legal regulations.

### Disposal

The product can be disposed of with the normal household waste.

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EN

### As-Basic Medical arm sling

Thank you very much for selecting a product of the Meyra-Ortopedia Group. The product complies with all legal regulations according to the Medical Products Law.

Please read the directions for use carefully and keep to the advice and recommendations given. Should you have further questions please address the responsible physician, your local specialist dealer or our service hotline.

### Product description

The product is used for the immobilisation of shoulder and upper arm as a postoperative treatment. The forearm steadily rests in a padded arm sling which is fixed around shoulder and waist with quick-lockbuckles. For a higher convenience the shoulder strap is padded.

### Safety instructions

The first application of the product must be carried out by special personnel trained on these kind of products as otherwise the product liability cannot be guaranteed. Basically, when adjusting the Velcro straps it has to be ensured that they are not fastened too tight to avoid constrictions. The product must not be worn on open wounds.

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Indications

The arm sling is used for the postoperative immobilisation of the shoulder joint. Any other employment, i.e. departing from the direction, results in an expiration of the product liability.

Application

- 1. Adjust the shoulder strap to wide length and close it.
- 2. Lay the forearm into the arm rest, taking the thumb into the boy sidel loop and the fingers into the loop on the body far side.
- 3. Place the shoulder loop over head and side of the healthy shoulder and adjust them for a good fit without constrictions.
- 4. The forearm strap is laid around the waist of the healthy side and fastened on the rear side and of the armrest and adjusted to the required length.

Contraindications:

If the product is employed properly contradictions are not known. If the slings are too tight pressure symptoms may evolve.

Wear time

Wear time is determined in consultation with your doctor.

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Further information

- 1. The product should only be worn under the care of a physician and for the intended clinical pictures.
- 2. The product is intended for the treatment of one patient.
- 3. Improper application excludes the product liability of the manufacturer.
- 4. Should the symptoms aggravate a physician should be consulted immediately.
- 5. A combination with other products has to be agreed with the physician in advance

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