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Storage information

The storage location should be dry and at room temperature. The product has to be protected against direct sunlight exposure and humidity.

Cleaning information

Please hang up the bandage dripping wet.

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Guarantee

The guarantee lies within the framework of the legal regulations.

Disposal

The product can be disposed of with the normal household waste.

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EN

Epi Medical elbow bandage

Thank you very much for selecting a product of the Meyra-Ortopedia Group. The product complies with all legal regulations according to the Medical Products Law.

Please read the directions for use carefully and keep to the advice and recommendations given. Should you have further questions please address the responsible physician, your local specialist dealer or our service hotline.

Product description

The Epi medical active bandage is a highly effective medical bandage on the latest technical level. The anatomically bi-elastic knitted form combined with the gel pads provides a very good fit as well as high support and safety in the movement. The tendon insertion in the elbow is effectively relieved, the pain eases. The integrated elastic gel pads increase the compression and have a massage effect which supports the blood circulation and contribute to detumescence, relief of pain and function improvement.

Safety instructions

The first application of the product must be carried out by special personnel trained on these kind of products as otherwise the product liability cannot be guaranteed.

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Employment according to purpose and indications

The Epi medical active bandage is used in case of irritations of the tendon insertion (tennis elbow, golfer's elbow), joint attrition, arthritis, irritations and muscular tendon diseases. Any other employment, i.e. departing from the direction, results in an expiration of the product liability.

Application of the bandage

1. Turn the upper edge down to the first third of the gel pads.
2. Pull the bandage on the forearm up to the elbow.
3. Now turn up the upper part of the bandage.
4. Adjust the bandage until the gap of the gel pad lies on the outer epicondylus of the elbow. Ideally, the inner epicondylus should lie in the gap of the lower gel pad.
5. To remove the bandage follow these steps
6. vice versa.

Contraindications

Provided that the product is employed properly contraindications are not know. If the bandage is too tight pressure symptoms may evolve. In case of the following clinical pictures the product must only applied after agreement with the responsible physician: skin changes or inflammations in the area concerned, scarring, hypersensitivity because of heat production, varicose veins, circulatory disturbance in arms and hands, lymphatic drainage disturbance.

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Application duration

The bandage should mainly be worn during physical activity as the function only works in this state. When resting (sleeping, sitting) the product should be removed.

Further information

- The bandage should only be worn under the care of a physician and for the intended clinical pictures.
- The Epi medical active bandage is intended for the treatment of one patient.
- Improper application excludes the product liability of the manufacturer.
- Should the symptoms aggravate a physician should be consulted immediately.
- A combination with other products has to be agreed with the physician in advance.

Available sizes

Circumference measured 12 cm below elbow joint

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