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## Storage information

The storage location should be dry and at room temperature. The product has to be protected against direct sunlight exposure and humidity.

## Cleaning information

Washing by hand in lukewarm water with mild detergent. Please hang up the bandage dripping wet.

## Material

Polyester, aluminium.

## Guarantee

The guarantee lies within the framework of the legal regulations.

## Disposal

The product can be disposed of with the normal household waste.

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## Manu Medical Plus wrist orthosis with thumb hold

Thank you very much for selecting a product of the Meyra-Ortopedia Group. The product complies with all legal regulations according to the Medical Products Law.

Please read the directions for use carefully and keep to the advice and recommendations given. Should you have further questions please address the responsible physician, your local specialist dealer or our service hotline.

## Product description

The Manu medical Plus wrist orthosis is a highly effective orthosis on the latest technical level. The material is permeable to air which creates a comfortable microclimate and high wearing comfort. The three Velcro fastening combined with the anatomically preformed metal insert allow a steady immobilization and stabilization of the wrist. The immobilization of the thumb metacarpophalangeal joint is realized by means of Velcro fastening.

## Safety instructions

The first application of Manu medical Plus must be carried out by special personnel trained on these kind of products as otherwise the product liability cannot be guaranteed. Basically, when adjusting the Velcro straps it has to be ensured that they are not fastened too tight to avoid congestion.

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## Employment according to purpose and indications

The Manu medical Plus is used in case of soft-tissue irritations of the wrist and the thumb metacarpophalangeal and thumb saddle joint, for the conservative immobilization after distortions and in case of tendovaginitis, carpal tunnel syndrome and rheumatoid arthritis. Any other employment, i.e. departing from the direction, results in an expiration of the product liability.

## Application of the bandage

1. Close all three Velcro straps at the outer ends and pull the orthosis over hand and wrist.
2. Firstly tighten the upper strap at the back of the hand, then the middle strap around the wrist and then the outer strap. At last fix the thumb with a Velcro strap.
3. The orthosis should have an evenly tight fit without constrictions. If necessary, readjust the Velcro straps.

## Contraindications

Provided that the product is employed properly contraindications are not known. If the orthosis is too tight pressure symptoms may evolve. In case of the following clinical pictures the product must only be applied after agreement with the responsible physician: skin changes or inflammations in the area concerned, scarring, hypersensitivity because of heat production, varicose veins, circulatory disturbance in the hands and arms, lymphatic drainage disturbance.

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## Application duration

The orthosis should mainly be worn during physical activity as the function only works in this state, but also for immobilization when resting.

## Further information

- The orthosis should only be worn under the care of a physician and for the intended clinical pictures.
- The Manu medical Plus is intended for treatment of one patient.
- Improper application excludes the product liability of the manufacturer.
- Should the symptoms aggravate a physician should be consulted immediately.
- A combination with other products has to be agreed with the physician in advance.

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