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EN

Prohand **orthosis stabilizing the wrist joint**

Thank you very much for selecting a product of the Meyra-Ortopedia Group. The product complies with all legal regulations according to the Medical Products Law.

Please read the directions for use carefully and keep to the advice and recommendations given. Should you have further questions please address the responsible physician, your local specialist dealer or our service hotline.

Product description

Elastic, breathable fabric provides optimal wrist compression, stabilization and prevents oedema. Prohand is available in one universal version, suitable for right and left hand. Due to simple construction it is easy to use and maintains proper medical and functional characteristics.

Safety instructions

The first application of the product must be carried out by special personnel trained on these kind of products as otherwise the product liability cannot be guaranteed. The product must not be worn on open wounds.

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Indications:

Follow-up treatment after wrist injuries. Wrist inflammation.
Pain syndromes of different etiology. Degenerative wrist disorders.
Wrist protection while taking up sport and professional activities after injury.

Directions for use

- » Prohand should be used according to doctor's/physiotherapist's advice.
- » One universal size is suitable for every wrist circumference.
- » For the first time, Prohand should be put on with the help of medical specialist who will give you all necessary instructions.
- » While putting the orthosis make sure that the thumb is placed in the thumb hold.
- » Then, enfold the band around the wrist. Start from palmar part, then continue to the outside part of the palm. Fasten with Velcro band.

Mechanism of acting

Construction provides optimal wrist compression, improves joint stabilization and relieves muscle-ligament structures. Properly shaped thumb hold enables unhampered movements.

Wear time

Wear time is determined in consultation with your doctor.

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Guarantee

The guarantee lies within the framework of the legal regulations.

Disposal

The product can be disposed of with the normal household waste.

Wrist orthosis stabilizing

PROHAND wrist support provides optimal wrist joint stabilization, prophylaxis of injuries and overstrain. It effectively assists follow-up treatment and protects wrist while working, practicing sport, and during other activities which generate significant load. High-quality fabric ideally adapts to your body and ensures comfort of use.

Storage information

The storage location should be dry and at room temperature. The product has to be protected against direct sunlight exposure and humidity.

Cleaning information

Wash by hand using gentle detergents in temperature max 35 ° C. Do not chlorinate, do not dry clean, wring gently before drying. Do not dry close to the artificial warmth source. Do not iron.