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EN

Talu Medical **stabilizing ankle joint orthosis**

Thank you very much for selecting a product of the Meyra-Ortopedia Group. The product complies with all legal regulations according to the Medical Products Law.

Please read the directions for use carefully and keep to the advice and recommendations given. Should you have further questions please address the responsible physician, your local specialistdealer or our service hotline.

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Product description

Orthosis consists of two shins made of plastic, which cover medial and lateral ankle. These are padded with soft foam and stabilizing bands which enables proper placing and stabilization of ankle joint. Adjustable plantar band provides circumferential regulation of the orthosis and movements of ankle joint plantar and dorsal flexion.

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Safety instructions

The brace should be used according to the instructions given by the doctor or physiotherapist.

When used for the first time, the support should be put on by the experienced health-care personnel.

Before the first use, please, read the user's manual carefully.

If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

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Indications:

Post-traumatic conditions of ankle joint (sprain, dislocation)during functional treatment. Instability of ankle joint.
Stabilization after surgeries. Ankle joint protection from recurrent injuries while re-undertaking occupational or sport activities after experiencing injury.

Application

Never put the brace on a bare foot (the best option is to put the brace on a soft, cotton sock) which allows to avoid skin irritation and abrasion.

1. Unfasten the stabilizing bands.
2. Put on the brace so that the junctive band would be under the foot and two parts of brace embrace the lateral ankle and medial ankle (using a Velcro band adjust the width of junctive band).
3. Make sure that the brace is placed parallel to longitudinal axis of lower limb.
4. Fasten the stabilizing bands around the ankle joint so that the brace would adhere to the body but would not restrict circulation.
5. Place feet with the orthosis inside shoe. While using the orthosis wear comfortable, sport shoes.

Contraindications

Conditions which require immobilization of ankle joint. Difficult to heal wounds, sores and other skin changes. Dysesthesia

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Further information

- The product should only be worn under the care of a physician and for the intended clinical pictures.
- The product is intended for the treatment of one patient.
- Improper application excludes the product liability of the manufacturer.
- Should the symptoms aggravate a physician should be consulted immediately.
- A combination with other products has to be agreed with the physician in advance

Guarantee

The guarantee lies within the framework of the legal regulations.

Disposal

The product can be disposed of with the normal household waste.

Storage information

The storage location should be dry and at room temperature. The product has to be protected against direct sunlight exposure and humidity.

Cleaning information

Clean the brace with moist cloth using mild detergents.