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EN

The decision as to when apply this brace is at the discretion of the physician or healthcare professional. Please read these instructions completely to help provide proper it.

Indication:

Solid Walker orthosis provides stabilization and protection in the case of soft tissue injuries around the ankle joint, healing of table fractures around the ankle and metatarsus.

Contraindications:

The air chambers should not be excessively pumped, this may lead to skin irritation. Attention should be paid to patients with sensory or diabetic disorders. Skin changers should be controlled if thy occur, and the air level in the chambers should be reduced. The orthosis should not be used in case of skin ulceration or open wounds. In case of numbness or pain symptoms within the orthosis, you should contact your doctor immediately.

Application:

1. Choose the correct walker size for the patient. The end of the patient s toes should not extend past the toe of the walker.
2. Remove the anterior (front) plastic panel from the walker. Open the hook and loop closures on the walker liner in preparation for placing the patient foot in the liner. Place the sock on the walker patient s foot.
3. Position the patient' s foot in the walker liner, sliding the patient s' heel in proper position as far towards the heel area of the walker as possible.
4. Secure the liner around the patient s foot and lower leg with the hook and loop closures. Attach the toe cover by folding it over the toes and securing the hook and liner. Note: The toe cover can be removed if desired.
5. Position the anterior plastic panel over the walker liner with the hook tabs on the panel holding it in place.
6. Beginning with the toe strap, feed the straps through the loops and secure the hook and loop closures. The straps should then be adjusted to ensure a secure and comfortable it. Caution should be taken to not over tighten the straps.
7. The inflation tubes are located on the sides of the walker and can be re-positioned on the liner if desired.
8. Inflate the air liner chambers by placing the CLEAR end of the inflation bulb into the air tubes and pump with one hand. Both air chambers should be inflated to provide even compression.

9. Caution: DO NOT OVER INFLATE THE AIR CHAMBERS.
10. To release air from the chambers, place the BLACK end of the iniation bulb into the air tubes and pump to deflate.
11. The air chambers and in Flation tubes can be removed from the liner if desired.

Use:

For hygienic and patient's comfort reasons, we recommend the lacing to be used on seamless cotton underwear. Do not forget to choose the size carefully, to ensure proper operation of the lacing.

1. Open the main Velcro of the lacing.
2. The lacing should be put on when the user is lying down, to allow stabilization of the sacro-lumbar section structures when relieved.
3. Put on the lacing so that its lower edge is about cm above the end of the coccyx (2 finger width above the cleavage), and the inner label is at the top edge.
4. The rear braces should adhere to the sacro-lumbar vicinity.Fasten the main Velcro of the lacing then fasten the tightening tapes, adjusting the pressure and level of the dorsum pressure of the corset.
5. The lacing should be fastened to ensure stabilization (partly limiting the spine movement within the sacro-lumbar section), but not to disturb breathing.
6. The time of wearing the lacing should be strictly determined by the physician or a physiotherapist. A rehabilitation training programme should be implemented in parallel.

Warring!

This product is designed for single patient use. The walker should be worn with a sock. To prevent irritation or rash, the liner or sock can be hand washed in mild soap and rinsed thoroughly. AIR DRY ONLY. Additional pads are included if needed.