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EN

⚠ ATTENTION:

Read all instructions, warnings and precautions before use. The correct application is necessary for the proper functioning of the brace. The brace should always be used as prescribed by a doctor or physiotherapist. For the first time, an orthosis should be fixed and adjusted by qualified medical personnel. The range of motion settings should be applied as directed by a physician or physiotherapist. Read the instructions carefully before using the brace. If you experience discomfort, swelling or skin irritation when using the brace, contact your doctor or physiotherapist immediately.

INDICATIONS:


- Mild to severe unicompartmental knee osteoarthritis with ligament instabilities,
- increased rehabilitative and functional support for diagnoses that require reduced medial or lateral joint loads,
- for daily activities.

CONTRAINDICATIONS:


Advanced degenerative changes involving both compartments of the knee, conditions requiring immobilization of the knee, wounds, ulcers, skin lesions within the knee brace area. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. The Oasis Brace was designed to provide maximum comfort for the patients with single-compartment gonarthrosis. The brace can be perfectly adjusted to the individual anatomical shape of the leg while applying corrective force. Flexion and extension stop adjustments to the hinge should be made only by, or under the advice of, your physician.

INSTRUCTION FOR USE:


1. Release each of the straps, attach to its loosening setting if it is the first time to apply the brace.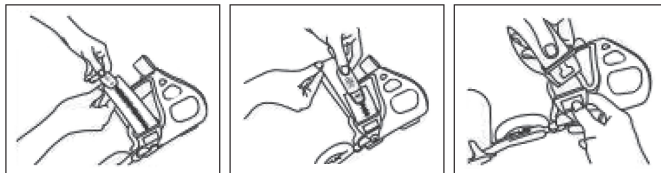


2. Detach all buckles.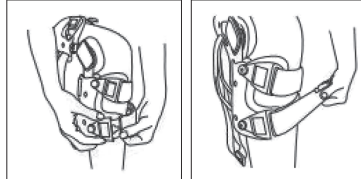

3. Apply the knee brace with the patient seated and knee joint flexion at 80° - 90°.
4. Press the shell against the leg, secure number 1 buckle around the calf. Adjust the strap length to desired fit by using the hook and loop closure. Fasten strap number 2 around the lower calf muscles.
5. Fasten the thigh straps in proper order and in the same way as strap number 1.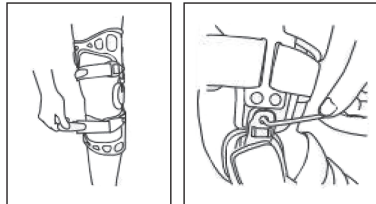

6. Stand and check the tightness of the straps.
7. Walk with the brace on to see if the strap causes any discomfort or not, or cause the pain worseing. If it does, re-adjust the strap while seated. Then stand and walk again to test the pain relief.
8. Adjust the hinge with next wrench or minor adjustments.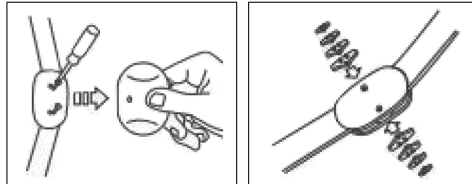

HOW TO SET HINGE:

1. Use a screwdriver to remove screws and detach the cover of hinge.
2. Take off the existing stop and insert new plug-in stops, which are desired flexion/extension degree.
3. Put the cover of hinge and screws back. Be sure that screws are seated and snug. Both medial (inside) hinge and lateral (outside) hinge stops must be identical in degrees.

⚠ ATTENTION:

Gradually increase brace wear time to allow your body to adjust to the brace. Start with 30 minutes on day one and gradually increase. Pain caused by osteoarthritis may take time to subside. It is important that

you routinely wear the brace during weight bearing activities (e.g. when on your feet) as it will reduce joint loading and allow you to begin feeling relief. It may take a number of weeks for relief to become significant.

Product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended.

THIS PRODUCT IS TO BE FITTED INITIALLY BY A PHYSICIAN (OR PROPERLY LICENSED PRACTITIONER) WHO IS FAMILIAR WITH THE PURPOSE FOR WHICH IT IS INTENDED.