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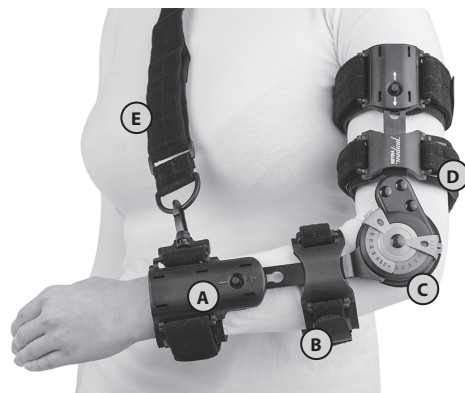
PLEASE SEEK MEDICAL ADVICE PRIOR TO USING THIS PRODUCT.

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EN

Product description:

One, universal model for left and right upper limb.
Universal size. Adjustable length from 34 to 43 cm.
A – button allows to adjust length
B – fabric pads prevent sliding and rotation of the orthosis and provide safe contact with the skin of the patient
C – hinge enable angular adjustment of motion or elbow joint immobilization.
D – malleable cuffs allows to adjust brace circumference
E – belt provides proper distribution of limb weight

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EN

Indications:

Treatment, follow-up treatment and rehabilitation of joint injuries which require limitation of pronation/ supination motions in forearm and motion control in elbow joint.

Contraindications:

Conditions which require complete immobilization of elbow joint, difficult to heal wounds, sores and other skin changes, strong spasticity.

Selection and fitting:

1. Locate the hinge pivot center over the lateral epicondyle (outer elbow) assess length. To adjust length move button A to “on” position and pull the cuff away from the elbow hinge to desire length and then move the button to “lock” position. The lower extension (forearm part) should extend up to the styloid process (wrist). The upper extension (upper arm part) should extend up to the most comfortable position at the arm pit.
2. If regulation the range of motion is necessary you should do it before putting on the brace. To set the range of motion push switch tab along the open direction to unlock the hinge. Set the flexion and extension pins to desire position and push switch tab back to lock and finish the setting.
3. Secure the straps, starting with the two strap closest to the elbow. Cuffs adjust to the limb circumference. If necessary, you should trim the straps for secure fit. Remove the hook piece, trim the strap and reapply the hook piece. Side bars may be bent to fit individual anatomic contour. Side bars must be bent till final length adjustment.
4. To take off the brace loosen all straps and cuffs.

Belt application:

1. Pass the end of the belt through the ring (next to the wrist strap) and fasten it with the velcro placed on the end of it.

2. Put the tape sequentially on the opposite arm to the injured limb, then on the back and then under the limb with orthosis.
3. Fasten carabiner on the end of belt to ring placed next to the wrist strap
4. The strap length should secure limb in comfortable position. If it is necessary the strap may be trimmed.

Maintenance:

Remove metal splint prior to washing. The straps and fabric pads should be washed up by hand using mild detergents. Do not chlorinate. Do not whirlpool, wring gently before drying. Dry in horizontal position. Dry in a distance of artificial sources of heat. Do not iron.

ATTENTION!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

ATTENTION!

The brace should be used according to the instructions given by the doctor or physiotherapist. When used for the first time, the support should be put on by the experienced health-care personnel. Range of motion should be adjusted according to instructions given by the doctors or physiotherapist. Before the first use, please, read the user's manual carefully. If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.