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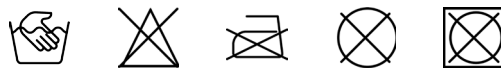
## Storage information

The storage location should be dry and at room temperature. The product has to be protected against direct sunlight exposure and humidity.

## Cleaning information

Elbow Safe should be washed using gentle detergents in max. temperature 35°. Do not chlorinate, do not dry clean, wring gently before drying. Do not dry close to the artificial warmth source. Dry in horizontal position. Do not iron.

## Guarantee

The guarantee lies within the framework of the legal regulations.

## Disposal

The product can be disposed of with the normal household waste.

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(EN)

### Elbow Safe Elbow brace

Thank you very much for selecting a product of the Meyra-Ortopedia Group. The product complies with all legal regulations according to the Medical Products Law.

Please read the directions for use carefully and keep to the advice and recommendations given. Should you have further questions please address the responsible physician, your local specialist dealer or our service hotline.

### Product description

Highly elastic, breathable fabric ensures optimal adaptation to anatomical shape soft tissue compression which prevents oedema. Additional pads and band relieve muscle- tendon structures, especially attachments of muscles extensor and flexor. Elbow Safe is an universal aid for right and left elbow.

### Safety instructions

The first application of the product must be carried out by special personnel trained on these kind of products as otherwise the product liability cannot be guaranteed. The product must not be worn on open wounds.

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Indications

Follow-up treatment of elbow joint injuries. Prophylaxis against overstrains of elbow joint.  
Follow-up treatment of tennis elbow and golfer’s elbow. Elbow joint pain syndromes of different etiology.  
Soft tissue disorders.

Directions for use

Elbow Safe should be used according to doctor’s or physiotherapist’s advice.  
Please, make sure to choose an appropriate size – measure the forearm circumference at its widest point. For the first time, the support should be put on in assistance of qualified medical personnel. Put the Elbow Safe so that the pads would be placed below the elbow joint (below the humeral bone epicondylus). Fasten the band and regulate the pressure according to doctor’s advice.

Wear time

Wear time is determined in consultation with your doctor.

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Mechanism of acting

Special construction provides optimal compression which improves elbow joint stabilization and relieves muscle-ligament structures. Additional pads and band relieve muscle attachments.

Elbow Safe

Is an elbow support of wide therapeutic use. It effectively helps rehabilitation, follow-up treatment of elbow injuries and dysfunctions. It provides optimal protection and overstrain prophylaxis. Additional pads and band relieve attachments of muscles extensor and/or flexor of carpal joint.  
Special construction ideally adapts to anatomical shape of joints and ensures optimal, therapeutic compression of softissue which prevents oedema and provides comfort of use.

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