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EN

Indications:

Disturbances in cervical spine stability after traumatic fractures, sprains and pathological fractures in the course of inflammatory or cancer treatment process. Necessity to restrict free movements of the cervical spine during fractures of the vertebral bodies. Pain syndromes of various etiology.

Contraindications:

Unstable fractures and neurological disorders. Clinical conditions that require lateral bending restriction.

Selection and fitting:

Proper application of the collar is as important as proper sizing for immobilization and patient comfort. The two-piece design consists of a front piece and a back piece which are packaged as a set.

Height and circumference measurement:

Measure the vertical distance from the top of the patient's shoulder to the tip of the chin. The measurement corresponds to the height of the collar. If the patient's measurement falls between two consecutive sizes, apply the smaller size first.

Measure around the patient's neck. The measurement corresponds to the size circumference of the collar.

- After selecting the properly sized collar, apply the back piece of the collar to the back of the patient's neck. Center the collar. The back arrow should point upward.
- Apply the frontpiece of the collar with the chin secured in the recess. Center the collar to secure neutral alignment. The front piece overlaps the back piece to ensure effective immobilization and comfort. The front arrow should point upward.
- With hook and loop fasteners, tighten the collar with a bilateral adjustment. This will secure the patient's cervical region in neutral alignment.

Maintenance:

Collar can be washed by hand in a very mild detergents at temperatures up to 35°C. Do not chlorinate, do not dry clean, do not wring, do not spin, squeeze gently before drying. Dry away from artificial sources of heat in spread form.

CAUTION:

This product is intended for individual use of each patient. If discomfort, swelling or skin irritation occurs during the use of the product, immediately contact your professional medical personnel. Proper selection of the collar is very important for immobilizing the patient's neck and comfort of use. Always set the patient's head in a neutral position.

CAUTION:

The final decision concerning the usage and selection of the appliance shall be made by the doctor or medical specialist.

Size:

Size	5,71 cm	8,25 cm	10,79 cm	13,33 cm
S (25-33)	S2	S3	S4	S5
M (33-40)	M2	M3	M4	M5
L (40-48)	L2	L3	L4	L5
XL (48 and more)	XL2	XL3	XL4	XL5