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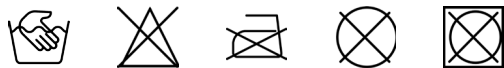
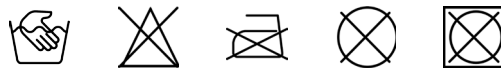
## Storage information

The storage location should be dry and at room temperature. The product has to be protected against direct sunlight exposure and humidity.

## Cleaning information

The Velcro fastening should be closed before washing. Please hang up the bandage dripping wet.

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### Lumbo Medical Pro lumbar bandage with stabilization struts

Thank you very much for selecting a product of the Meyra-Ortopedia Group. The product complies with all legal regulations according to the Medical Products Law.

Please read the directions for use carefully and keep to the advice and recommendations given. Should you have further questions please address the responsible physician, your local specialist dealer or our service hotline.

### Product description

The Lumbo medical Pro active bandage is a highly effective medical product on the latest technical level. The anatomically knitted form offers a very good fit and high wearability. The compression of the bandage stimulates the metabolism and decreases muscle tension which brings relief and alleviation of pain. The four integrated stabilization struts provide additional stability and support an anatomically correct posture.

### Safety instructions

The first application of Lumbo medical active bandage must be carried out by special personnel trained on these kind of products as otherwise the product liability can not be guaranteed. Basically, when adjusting the Velcro straps it has to be ensured that they are not fastened too tight to avoid congestion.

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Further information

- The active bandage should only be worn under the care of a physician and for the intended clinical pictures.
- The Lumbo medical pro is intended for the treatment of the patient.
- Improper application excludes the product liability of the manufacturer.
- Should the symptoms aggravate a physician should be consulted immediately.
- A combination with other products has to be agreed with the physician in advance.

Guarantee

The guarantee lies within the framework of the legal regulations.

Disposal

The product can be disposed of with the normal household waste.

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Employment according to purpose and indications

The Lumbo medical Pro active bandage is used in case of lumbagos caused by muscle debility, irritations of the intervertebral joints, symphysiolysis pain and ligamentosis. Any other employment, i.e. departing from the direction, results in an expiration of the product liability.

Application of the bandage

1. Lay the bandage around the lumbar area and position the strut inserts of the bandage in the middle of the back.
2. Now pull the left side to the centre of the body and fix the right side on top of it by means of the Velcro fastening.
3. The bandage should have a steady, tight fit avoiding constrictions.
4. If necessary, readjust the fit of the bandage.

Contraindications

Provided that the product is employed properly contraindications are not known. If the bandage is too tight pressure symptoms may evolve.

Application duration

The wearing period has to be agreed with the responsible physician.