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Storage information

The storage location should be dry and at room temperature. The product has to be protected against direct sunlight exposure and humidity.

Cleaning information

Remove the pad before washing. Please hang up the bandage dripping wet.

Material

Textile: Polyester, monofilament, elastane

Disposal

The product can be disposed of with the normal household waste.

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Lumbo Medical Light lumbar support bandage with stabilization struts

Thank you very much for selecting a product of the Meyra-Ortopedia Group. The product complies with all legal regulations according to the Medical Products Law.

Please read the directions for use carefully and keep to the advice and recommendations given. Should you have further questions please address the responsible physician, your local specialist dealer or our service hotline.

Product description

The Lumbo medical light active bandage is a highly effective medical product on the latest technical level. The anatomically knitted form offers a very good fit and high wearability. The compression of the bandage stimulates the metabolism and decreases muscle tension which brings relief and alleviation of pain. The four integrated stabilisation struts provide additional stability and support an anatomically correct posture. Safety instructions
The first application of Lumbo medical light active bandage must be carried out by special personnel trained on these kind of products as otherwise the product liability cannot be guaranteed. Basically, when adjusting the Velcro straps it has to be ensured that they are not fastened too tight to avoid congestion.

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Employment according to purpose and indications

The Lumbo medical Light is used in case of general backache, debility of the back muscles, lumbagos, irritations of the iliosacral joints, attrition of the spinal column. Any other employment, i.e. departing from the direction, results in an expiration of the product liability.

Application of the bandage

1. Lay the bandage around the lumbar area and hold the left loop of the bandage in the stomach region.
2. Now pull the right loop over the left and fix the bandage by means of the Velcro fastening.
3. The bandage should have a steady, tight fit avoiding constrictions.
4. If necessary, readjust the fit of the bandage.
5. The metal struts can be adjusted in the back area if required. In this case please contact specialist dealer or the responsible physician.
6. The exchange of the centre metal struts by plastic struts must only be carried out by authorised personnel or the responsible physician.

Contraindications

Provided that the product is employed properly contraindications are not known. If the bandage is too tight pressure symptoms may evolve.

Application duration

The success of the bandage can be directly connected with the wearing period. According to latest investigations a permanent application does not weaken the muscles.

Employment according to purpose and indications

The use of Lumbo medical Light dressing is recommended in the case of general back pain and muscle weakness.

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Further information

- The active bandage should only be worn under the care of a physician and for the intended clinical pictures.
- The Lumbo medical light is intended for the treatment of one patient.
- Improper application excludes the product liability of the manufacturer.
- Should the symptoms aggravate a physician should be consulted immediately.
- A combination with other products has to be agreed with the physician in advance.

Guarantee

The guarantee lies within the framework of the legal regulations.

Wielkość/ Size	Obwód w pasie/ Waist circumference	Nr artykułu/ Part no.
1	64 – 72 cm	27 440 11
2	72 – 80 cm	27 440 12
3	80 – 90 cm	27 440 13
4	90 – 100 cm	27 440 14
5	100 – 110 cm	27 440 15
6	110 – 120 cm	27 440 16